


# SORBACT® GEL DRESSING

Microbe binding moist wound-contact layer

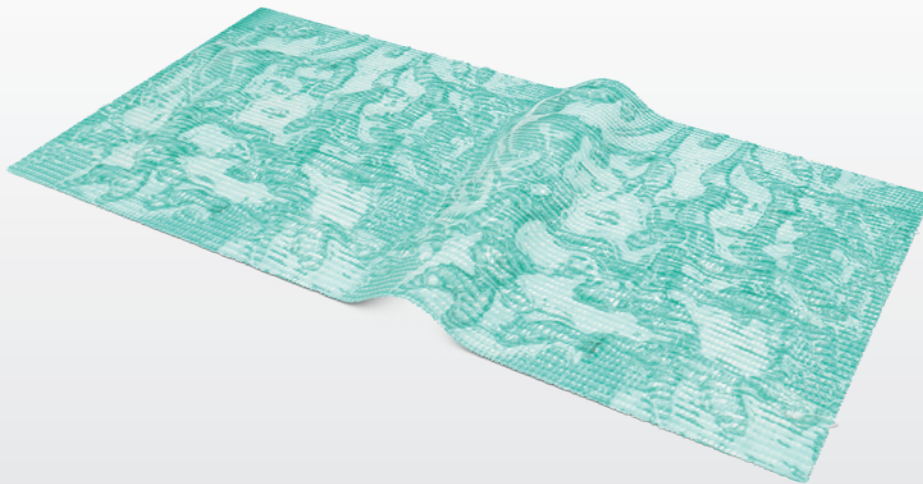

## REDUCES BIOBURDEN IN DRY TO LOW EXUDING WOUNDS

*Sorbact® Gel Dressing is a sterile, bacteria- and fungi-binding dressing covered with gel. It consists of a green Sorbact® wound-contact layer with a water-based gel containing carbomer and propylene glycol (10%). Sorbact® Gel Dressing donates moisture, enables a moist wound environment, and facilitates autolytic debridement.*

Product	Ref. no.	Size	Pcs/pack
Sorbact® Gel Dressing	98136	7.5 x7.5 cm, 3 x 3 in	10
Sorbact® Gel Dressing	98139	3 x 15 cm, 1.2 x 6 in	10
Sorbact® Gel Dressing	98137	7.5 x 15 cm, 3 x 6 in	10
Sorbact® Gel Dressing <b>NEW!</b>	98180	15 x 20 cm, 6 x 8 in	5
Sorbact® Gel Dressing <b>NEW!</b>	98181	20 x 25 cm, 8 x 10 in	5



## INTENDED USE

Sorbact<sup>®</sup> Gel Dressing is intended for use in the management of clean, contaminated, colonized or infected wounds with dry to low exudate levels, such as traumatic wounds, burns, cavity wounds, fistulas, pressure ulcers, diabetic ulcers and foot and leg ulcers.

## INSTRUCTIONS FOR USE

1. Prepare the wound according to local clinical practice.
2. Select an appropriate dressing size for the wound.
3. Remove the dressing from the pouch using an aseptic technique.
4. If cutting the dressing, use an aseptic technique. Discard any open and unused dressing.
5. Apply the dressing. Ensure that the dressing comes into direct contact with the entire wound surface. The dressing should not overlap the surrounding skin.
6. In cavity wounds and fistulas, always leave part of the dressing sticking out so that it can be easily removed.
7. Apply an appropriate secondary dressing.
8. The frequency of dressing changes depends on exudate levels and overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days.



*Clinicians should ensure that the product is used in accordance with the manufacturer's instructions for use.*

## SORBACT<sup>®</sup> RIGHT FROM THE START PREVENTS AND TREATS WOUND INFECTIONS

Sorbact<sup>®</sup> microbe binding dressings prevent and treat wound infections by lowering the bioburden. Microbes bind to the dressing surface and are removed when the dressing is changed.

Sorbact<sup>®</sup> is a Swedish innovation and the range of wound dressings is available through ABIGO partners and distributors in more than 65 countries worldwide. In some regions, Sorbact<sup>®</sup> is available as Cutimed<sup>®</sup> Sorbact<sup>®</sup> and Leukomed<sup>®</sup> Sorbact<sup>®</sup>. Find your local distributor at [abigo.com](http://abigo.com). Sorbact<sup>®</sup> is a registered trademark of ABIGO Medical AB.