


SORBACT[®] FOAM GENTLE BORDER

Microbe binding dressing with soft silicone border

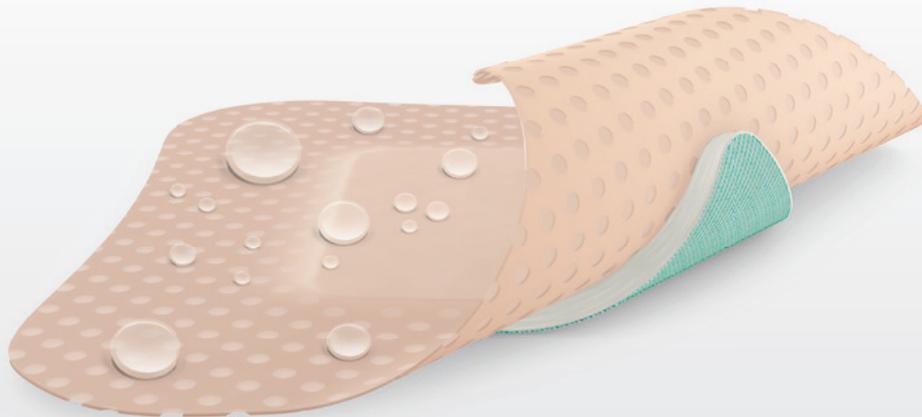

THE ATRAUMATIC ALL-IN-ONE DRESSING THAT REDUCES BIOBURDEN


Sorbact[®] Foam Gentle Border is a sterile, bacteria- and fungi-binding, self-adherent absorbent wound dressing. It consists of a green Sorbact[®] wound contact layer, absorbs and retains exudate, reducing the risk of maceration and ensuring a moist wound environment. The soft, skin-friendly, adhesive silicone borders enable atraumatic dressing changes. Sorbact[®] Foam Gentle Border provides optimal conditions for wound healing.

Product	Ref. no.	Size	Pad Size	Pcs/pack
Sorbact [®] Foam Gentle Border	98531	7.5x7.5 cm, 3x3 in	3.5x3.5 cm, 1.4x1.4 in	10
Sorbact [®] Foam Gentle Border	98532	10x10 cm, 4x4 in	5x5 cm, 2x2 in	10
Sorbact [®] Foam Gentle Border	98533	15x15 cm, 6x6 in	10.5x10.5 cm, 4.1x4.1 in	10



INTENDED USE


Sorbact[®] Foam Gentle Border is intended for use in management of clean, contaminated, colonized, or infected wounds with moderate exudate levels, such as surgical wounds, traumatic wounds, pressure ulcers, diabetic ulcers and foot and leg ulcers.

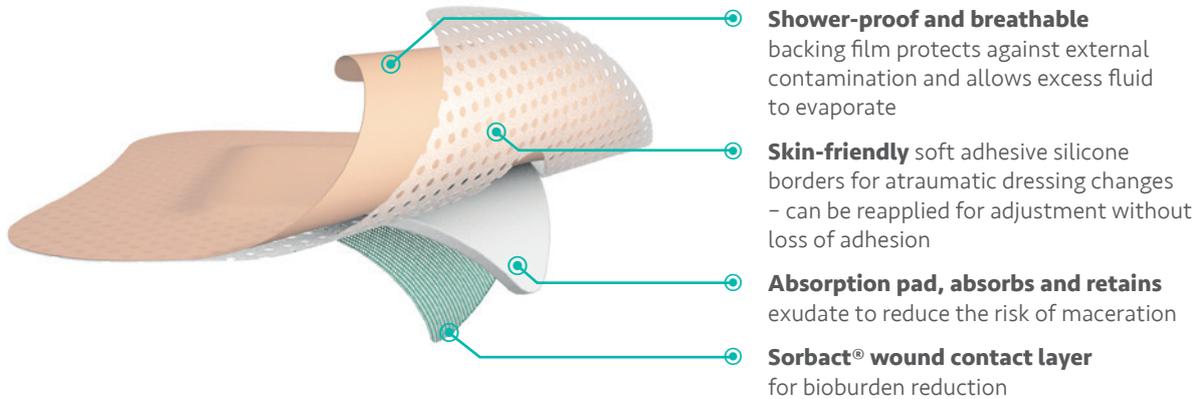

INSTRUCTIONS FOR USE

1. Prepare the wound according to local clinical practice. Ensure that the surrounding skin is clean and dry.
2. Select an appropriate dressing size for the wound. The wound pad should overlap the wound margins by 1 cm to minimize the risk of maceration.
3. Remove the dressing from the pouch using an aseptic technique.
4. Remove the protective film from the wound contact side of the dressing.
5. Apply the dressing. Do not stretch. Ensure that the green wound contact layer comes into direct contact with the complete wound surface.
6. Gently press the borders against the surrounding skin. If necessary, the dressing can be adjusted and reapplied.
7. The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days.

Suited for use in compression therapy.

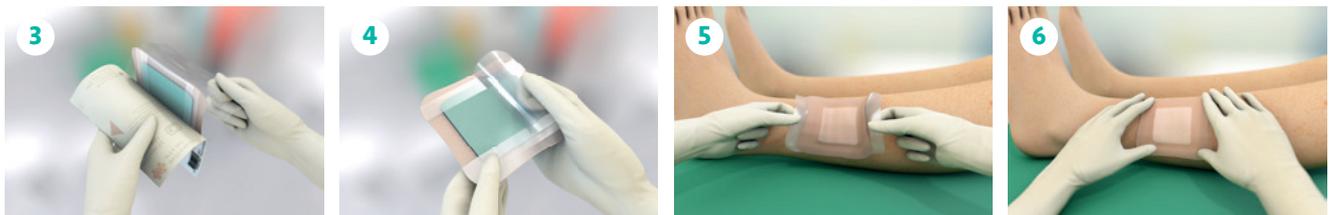

Clinicians should ensure that the product is used in accordance with the manufacturer's instructions for use.

SORBACT[®] RIGHT FROM THE START PREVENTS AND TREATS WOUND INFECTIONS

Sorbact[®] microbe binding dressings prevent and treat wound infections by lowering the bioburden. Microbes bind to the dressing surface and are removed when the dressing is changed.

Sorbact[®] is a Swedish innovation and the range of wound dressings is available through ABIGO partners and distributors in more than 65 countries worldwide. In some regions, Sorbact[®] is available as Cutimed[®] Sorbact[®] and Leukomed[®] Sorbact[®]. Find your local distributor at abigo.com. Sorbact[®] is a registered trademark of ABIGO Medical AB.