


# SORBACT<sup>®</sup> SUPERABSORBENT

Microbe binding highly absorbent dressing

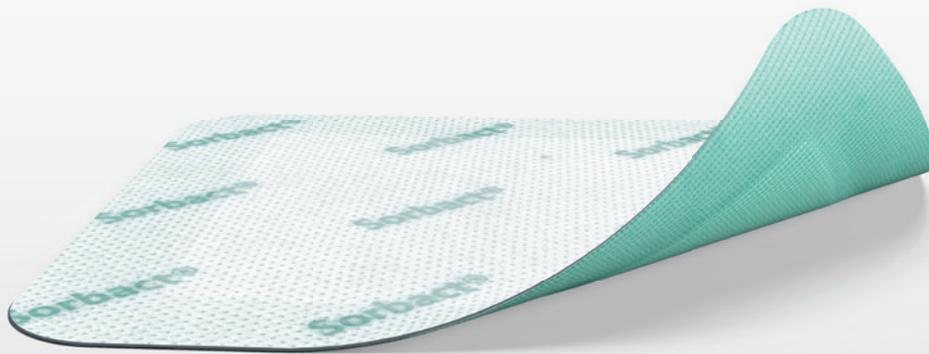

## REDUCES BIOBURDEN IN HIGHLY TO EXCESSIVELY EXUDING WOUNDS


*Sorbact<sup>®</sup> Superabsorbent is a sterile, bacteria- and fungi-binding superabsorbent wound dressing. It consists of a green Sorbact<sup>®</sup> wound contact layer combined with a super-absorbent core and a white water-repellent backing to prevent exudate strikethrough.*

Product	Ref. no.	Size	Pcs/pack
Sorbact <sup>®</sup> Superabsorbent	98501	10x10 cm, 4x4 in	10
Sorbact <sup>®</sup> Superabsorbent	98502	10x20 cm, 4x8 in	10
Sorbact <sup>®</sup> Superabsorbent	98503	20x20 cm, 8x8 in	10
Sorbact <sup>®</sup> Superabsorbent	98504	20x30 cm, 8x12 in	10



## INTENDED USE

Sorbact<sup>®</sup> Superabsorbent is intended for use in the management of clean, contaminated, colonized or infected wounds with high to excessive exudate levels, such as surgical wounds, traumatic wounds, pressure ulcers, diabetic ulcers and foot and leg ulcers.

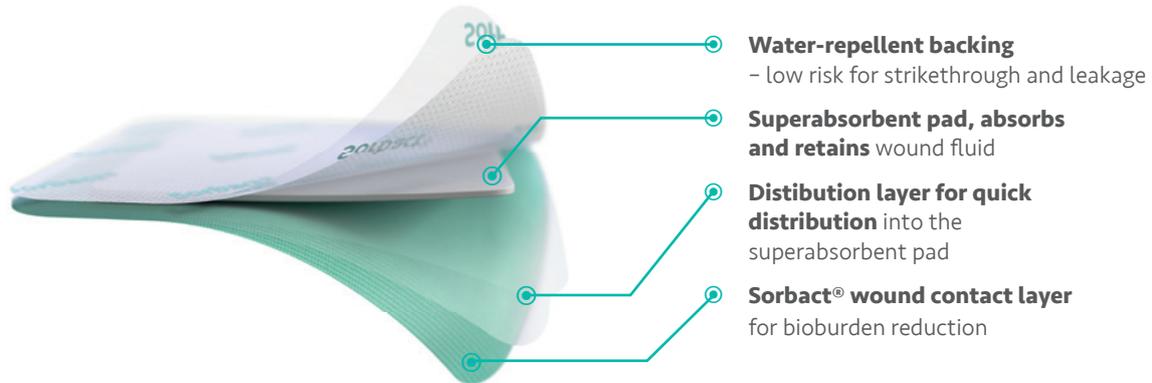

## INSTRUCTIONS FOR USE

1. Prepare the wound according to local clinical practice.
  2. Select an appropriate dressing size for the wound. The wound pad should overlap the wound margins by at least 2 cm.
  3. Remove the dressing from the pouch using an aseptic technique.
  4. Apply the dressing. Ensure that the green side of the dressing comes into direct contact with the entire wound surface.
  5. Fixate with tape at dressing edges or cover with bandage. Ensure the dressing is allowed to expand under the fixation.
  6. The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. If saturated the dressing shall be changed.
- Suitable for use in compression therapy.



Clinicians should ensure that the product is used in accordance with the manufacturer's instructions for use.

## SORBACT<sup>®</sup> RIGHT FROM THE START PREVENTS AND TREATS WOUND INFECTIONS

Sorbact<sup>®</sup> microbe binding dressings prevent and treat wound infections by lowering the bioburden. Microbes bind to the dressing surface and are removed when the dressing is changed.

Sorbact<sup>®</sup> is a Swedish innovation and the range of wound dressings is available through ABIGO partners and distributors in more than 65 countries worldwide. In some regions, Sorbact<sup>®</sup> is available as Cutimed<sup>®</sup> Sorbact<sup>®</sup> and Leukomed<sup>®</sup> Sorbact<sup>®</sup>. Find your local distributor at [abigo.com](http://abigo.com). Sorbact<sup>®</sup> is a registered trademark of ABIGO Medical AB.