

# Otinova<sup>®</sup> ear spray

MA-005-06-EN, 2018-02-24 ABIGO 2018-126

## Product information

20 ml ear-spray bottle containing 15 ml solution (150 spray doses).  
Shelf-life in room temperature: 3 years from manufacturing date (see date on package and bottle), opened bottle 6 months.

**Distributed** in Sweden by Tamro and Oriola.

**GTIN-13:** 7 30000902 863 4

## References

Available by Amellnova on request.

## Website

[www.otinova.com](http://www.otinova.com)

## Indication

Prophylaxis and treatment of inflammation of the ear canal (external otitis, otitis externa, swimmers ear).

## Product description

Otinova<sup>®</sup> ear spray is a scientifically and clinically documented medical device that efficiently and safely prevents and treats external otitis.

Otinova<sup>®</sup> ear spray is registered by the Medical Products Agency for the indication prophylaxis and treatment of external otitis. Otinova<sup>®</sup> is classified as a medical device, class I, according to rule 5 in Annex IX of the MDD 93/42/EEC.

Otinova<sup>®</sup> ear spray is sold by pharmacies.

## Content and mode of action

Otinova<sup>®</sup> contains aluminum acetate and aluminum acetotartrate (1,8% aluminum), acetic acid (8,25%), water, pH 3-4 (Burows solution).

Otinova<sup>®</sup> does not contain antibiotics or steroids.

Otinova<sup>®</sup> acts locally, is astringent, relieves itching and has an antibacterial and anti-fungal effect.

## Dosing and usage

Otinova<sup>®</sup> is sprayed 1-2 times into each ear, morning and evening and can be used by both adults and children above 5 years of age. It is important to tilt the head after spraying so that the solution reaches the innermost part of the ear canal.

Do not use Otinova<sup>®</sup> for a continuous period of more than 7 days.



## ABIGO Medical AB

Ekonomivägen 5, 436 33 ASKIM, Sweden  
Tel +46 31 748 49 50  
[pharma@abigo.se](mailto:pharma@abigo.se) [www.abigo.se](http://www.abigo.se)

## AMELLNOVA AB

Part of the Abigo Group